

Technical Specification Sheet

3M™ Particulate Respirator 8210, N95

Key Features

- NIOSH approved N95 rating
- Adjustable nose clip
- Nose foam
-

Ultrasonically welded headbands

Material Composition


- Straps – Thermoplastic Elastomer
- Nose Clip – Aluminum
- Nose foam - Polyurethane
- Filter – Polypropylene
- Shell – Polyester
- Coverweb - Polyester
- This respirator contains no components made from natural rubber latex
- Approximate weight of product: 0.35 oz.

Country of Origin Made in the USA with globally sourced materials

Use For

- Use for solid particulates and liquid mists in concentrations not exceeding 10X PEL/OEL
- Always follow User Instructions and use in manners as indicated

Do Not Use For

- DO NOT use for gases and vapors, oil aerosols, asbestos, arsenic, cadmium, lead, 4,4- methylene dianiline (MDA), abrasive blasting
- DO NOT use for particulate concentrations exceeding 10X PEL/OEL
- DO NOT use in any manner not indicated in the User Instructions



Approvals and Standards

- NIOSH approved N95 particulate respirator
- Meets NIOSH 42 CFR 84 N95 requirements for a minimum 95% filtration efficiency against solid and liquid aerosols that do not contain oil.
- NIOSH approval number: TC-84A-0007
- Assigned Protection Factor (APF 10) per US OSHA and Canada CSA

Time Use Limitation Replace the respirator when it becomes dirty, damaged, or difficult to breathe through.

Shelf Life and Storage

- 5 years from the date of manufacture
- Use By date on box in 08/2024
- Store respirators in the original packaging, away from contaminated areas, dust, sunlight, extreme temperatures, excessive moisture, and damaging chemicals
- Store in temperatures between -4°F (-20°C) and +86°F (+30°C) and not exceeding 80% RH

WARNING!

This respirator helps reduce exposures to certain airborne contaminants. Before use, the wearer must read and understand the User Instructions provided as a part of the product packaging. Follow all local

regulations. In the U.S., a written respiratory protection program must be implemented meeting all the requirements of OSHA 1910.134, including training, fit testing and medical evaluation. In Canada, CSA standard Z94.4 requirements must be met and/or requirements of the applicable jurisdiction, as appropriate. Misuse may result in sickness or death. For correct use, consult supervisor and the User Instructions



Personal Safety Division
3M Center, Building 0235-2W-70

Acceptable Fit Test Protocols

Fit Test Protocol*		Acceptable with this product?
Qualitative Protocols	Saccharin	<input checked="" type="checkbox"/>
	Bitrex™	<input checked="" type="checkbox"/>
	Irritant Smoke	<input type="checkbox"/>
	Isoamyl Acetate	<input type="checkbox"/>
Quantitative Protocols		<input checked="" type="checkbox"/>

*Refer to OSHA 1910.134